

Instructions for use
Paco-Flow Orthopedic Suction Tip

The device is intended for the removal and collect of bone fragments, large clots , other debris during orthopedic procedures.

Indications and contraindications

This device is indicated as per clinical practitioner's professional determination. There is no specific contraindication for use of this device.

Warnings and Cautions

1. Inspect the product before use. If there is any sign of malfunction, do not use the product and replace it by a new one.
2. Only personnel instructed and trained in its use should operate this product, following operating procedures in this IFU.
3. Single use, re-use of this product will cause the risk of cross infection and unpredictable health threat. Discard the product per hospital's procedure.















Operation

1. Select the desired tip and attached to the handle.
2. Point the tip to the surgical site to remove the debris during operation.

Clinical claim

The device has been introduced into the market for a long time. To the best of manufacturer's knowledge, no side effect reveals on use of this device.

Symbols used for labeling

 REF	Catalogue number	 LOT	Lot number		Date of manufacture
	Used by		Do not use if package is damaged		Caution
	Do not reuse	 MD	Medical Device	 UDI	Unique Device Identifier
	Consult instructions for use	 STERILE EO	Sterilized using Ethylene oxide		Do not resterilize.
	Manufacturer	Rx ONLY			
	Temperature limitation				

 Pacific Hospital Supply Co., Ltd.
4F, No. 160, Daye Rd, Beitou Dist, Taipei 112, Taiwan

Note: Any serious adverse event occurring in relation to the device in question should be reported to the manufacturer and the competent local authority.